

Cleaning and Disinfection Guide

As influenza and epidemics become more frequent around the world than before, demands to prevent nosocomial infections are becoming important. In particular, cleaning and disinfecting medical devices are the most fundamental ways to prevent infections. We provide these guides to ensure our clients, patients, and clinicians to stay safe and to mitigate infections.

Fetal



ECOtwin LED / ECOtwin LCD / ECOsound / SMART 1 / SMART 3



ECG



CARDIO M / CARDIO M-PLUS / PHYSIO PORT
CARDIO M-HOLTER LZ EKG / CARDIO E3
CARDIO M-PC / CARDIO M-PAD 7 & 10



Monitoring



PROview 10L / PROview 10 / PROview 12 / M10 / M20 / M20|EKG



BDM



Excellus / Primus / inus|D / inus|S



! Recommended cleaners & disinfectants for medical ECONET devices

Cleaning and disinfection should be done in appropriate ways and agents. If medical devices are exposed to unapproved agents, medical-related infections or damages to the components of devices could occur.

To reduce possibility of device damage and the spread of pathogens, we recommend wipes -ready-to-use and alcohol-free.

So they can be used to disinfect the surfaces of alcohol-sensitive medical devices simply and efficiently. Liquid products supplied as a concentrate, should be properly adjusted according to the intended use.

Some products can be used with cloths or wipes.